

Silicone Pad

Materials

- Silicone mold kit
 - o [BBDINO Silicone Mold Kit](#)
 - o Any silicone mold making kit will work. Ordering in bulk may help reduce cost
- Setting pan – approximately 5x7”, cured silicone can easily be cut to desired size
 - o Please read manufacturer instructions. Some kits may require baking.

Steps

1. Mix the silicone according to manufacturer instructions
2. Pour silicone into your setting pan, in a layer approximately $\frac{1}{4}$ ” thick
3. Allow silicone to cure according to manufacturer instructions (4-24 hours depending on brand)
4. Once silicone is completely cured, remove from pan & cut to the size of the PVC tube used in “02 Vagina Model Assembly” (approximately 3” in diameter)
 - Save the extra “scrap” strips of silicone to be used later!
5. Cut an “X” into the middle of the silicone pad – see picture



Silicone pad with PVC vagina. Use PVC tube to trace a circle on the silicone pad



Silicone pad cut down to size



Silicone pad with an X cut in the middle